Sol - Guard[®] **Safety Insulin Syringe** **with Fixed Needle**

Instructions for Use

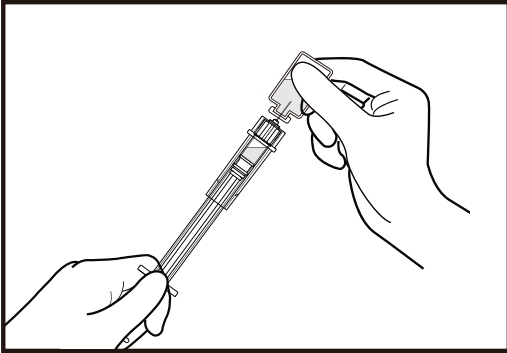
Figure 1

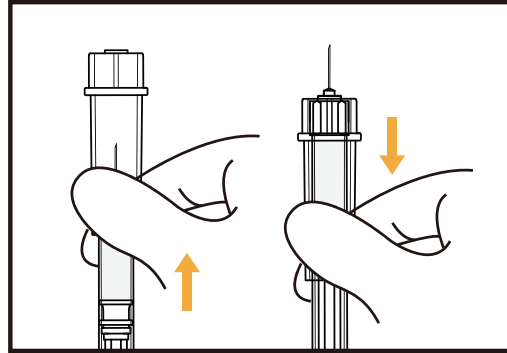
Figure 2

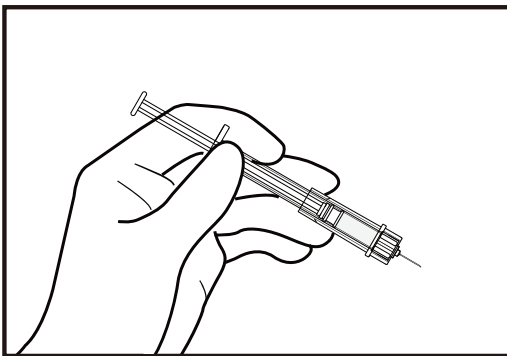
Figure 3

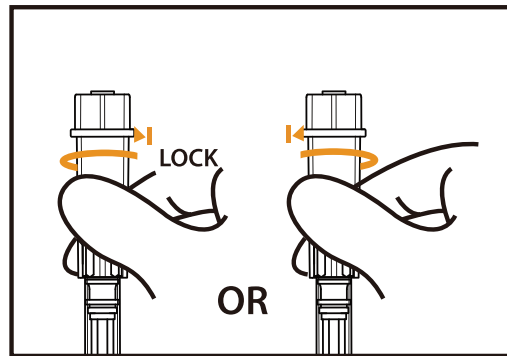
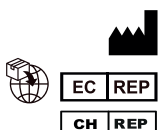
Figure 4

 Sol-Millennium Medical, Inc.
 311 S. Wacker Drive, Suite 4100, Chicago, Illinois, 60606, USA
 Sol-Millennium Europe Sp. z o.o. Twarda 18, 00-105 Warsaw, Poland
 Sol-Millennium Swiss R&D Center SA
 Piazzetta San Carlo 2, 6900 Lugano, Switzerland

UK Sol-Millennium UK Ltd.
Responsible Person: Paternoster House, 65 St. Paul's Churchyard, London EC4M 8AB, United Kingdom

Sponsor in Australia: Sol-Millennium Medical Pty Ltd.
 Suite 14 26-30, Tedder Avenue, Main Beach QLD 4217, Australia

Malaysia Authorized Representative: Andaman Medical
 Bridge Sdn. Bhd.
 Unit 3.3A, 3rd Floor Wisma Leader, No.8 Jalan Larut,
 10050 Penang, Malaysia
<https://www.andaman-med.com>
 Registration No.: GB23186449417

PAP

2023/04
8101 REV1
Made in China

Rx only (USA)

CE 2797

SolM Building a Healthier Tomorrow

SAFETY SYRINGE WITH FIXED NEEDLE

EN

Indication for use

Used to withdraw and inject insulin into the body.

Intended users

Licensed health care professional (HCP) or self- injecting patients after proper training obtained from HCP.

Intended purpose

The Insulin Safety Syringe is intended for the delivery of U-100 insulin. In addition, the Insulin Safety Syringe Safety Sleeve covers the needle when activated. In the activated position, the Safety Shield guards against accidental needle stick.

Contraindications

There are no known contraindications.

Intended population

Anyone who require withdrawal and injection of medication (insulin) including adults and children.

Clinical Benefit

Delivery of indicated medication (insulin), prevention of accidental needle stick injury.

Performance Characteristics

Dose accuracy at nominal capacity: +/-5% in accordance with ISO 8537.

Mode of use

1. Peel package as indicated to allow proper removal of the Sol-Guard™ Safety Syringe with fixed needle and remove needle cap. Do not change the angle while withdrawing medication and removing needle from vial since it can bend and destroy the needle.
2. Prepare vial/ampoule for medication draw up.
3. Penetrate center of vial/ampoule with needle and draw up desired dose of medication. (See figure 1)
4. Extend safety shield forward to fully cover needle. This ensures the safety shield is ready for safe transportation to patient site. To give injection, slide back the safety shield until it snaps into its original position. (Figure 2 * For safe transportation option). Note: DO NOT twist safety shield or syringe. If the safety shield is locked by twisting, the device is unusable and should be discarded.
5. Administer medication according to your facility's established protocol. (Figure 3)
6. Immediately following injection, extend the safety shield forward fully. Next, to lock safety shield, twist in either direction until you feel the positive stop and hear an audible click. (Figure 4)
7. Dispose immediately used device into sharps collector.

⚠ Cautions

- This device is sterile unless the package has been damaged or opened.
- If unable to activate safety mechanism, discard syringe immediately into an approved sharps container.
- Single use device. Re-use or use if the package is damaged may lead to infection or other illness/ injury.
- Users must be trained by a healthcare professional and follow the Instructions For Use.
- Failure to follow these instructions could result in serious injury to the patient and/or clinician.
- In case that the safety mechanism is not activated by the end user properly- there is no negative impact on the medication dose delivered. The device must be disposed safely- immediately after use.
- Consult a healthcare provider about your medical condition and appropriate treatment.
- Always follow the medication manufacturer's instructions for use, contraindications, warnings and precautions."

Report any adverse event related to this device to the Manufacturer (complaints@sol-m.com) and to the local Competent Authority.

Federal (USA) law restricts this device to sale by or on order of a physician or other licensed practitioner. Follow standard infection control precautions as specified by the Centers Disease Control and Prevention (USA) or local equivalent. Not for drug storage, these syringes cannot be used as closed container for compounded and repackaged drugs. (Canada)

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